

TOTALIFT

Clinical TipSheet: Basic Operations

WY' EAST
MEDICAL



Stretcher



Chair



Lateral Transfer



Weight capacity
posted on base
400 lb



3 Positions
• Brake
• Free
• Steer



To adjust seat
position, squeeze red
handle and hold while
positioning chair



Release red handles
when chair is in
desired position

Accessories



O₂ Tank Holder



IV Pole Storage



IV Pole Placement

Hand Rail Instruction



1. Push Top of Rail in Toward
Center of Bed



2. Pull Black Lever Upward



3. Lower Side Rail

Foot/Leg Rest Instruction



1. Red Adjustment
Lever(Ironing Board Style)



2. Push Up and Slide Foot
Rest to Position Length



3. Squeeze Lever and Adjust
into Position

TOTALIFT

Clinical TipSheet: Lateral Transfer

WY'EAST
MEDICAL

1. Position Totalift in the stretcher position and parallel to the receiving surface; lock brakes.
2. Retrieve crank handle as seen here; turn lever until Totalift mattress/glide will be able to clear the second surface.



3. Position patient centered on mattress and in a side lying position facing away from the Totalift.
4. Turn until the Totalift mattress glides to the center of the second surface. Be careful not to over extend the mattress (refer to red lines printed on the frame). Tuck edge of mattress underneath patient.

5. Turn the patient to position onto the Totalift mattress, guide patient onto center of surface.



6. Once patient is centered on the mattress, turn in opposite direction to begin return movement.
7. Continue to turn until patient and mattress glide into place centered on Totalift chair.

8. Lift side rails until locked into position on both sides of the patient
9. Using red seat position handles, raise head to comfortable position as desired.



10. Ensure that hand crank is secured under the frame as seen here to allow for opposite side operations.



1-18015-R01

800.255.3126 | wyeastmedical.com