


LIMB SLING WITH LOOP STRAPS

Instructions For Use



Device Overview:

DEVICE NAME:	DESCRIPTION:	PART NUMBER:	MATERIAL:	SWL:
Limb Sling	Disposable Limb Loop Sling	52472 One Size	Cotton/Poly Blend	70 kg. / 154 lbs.

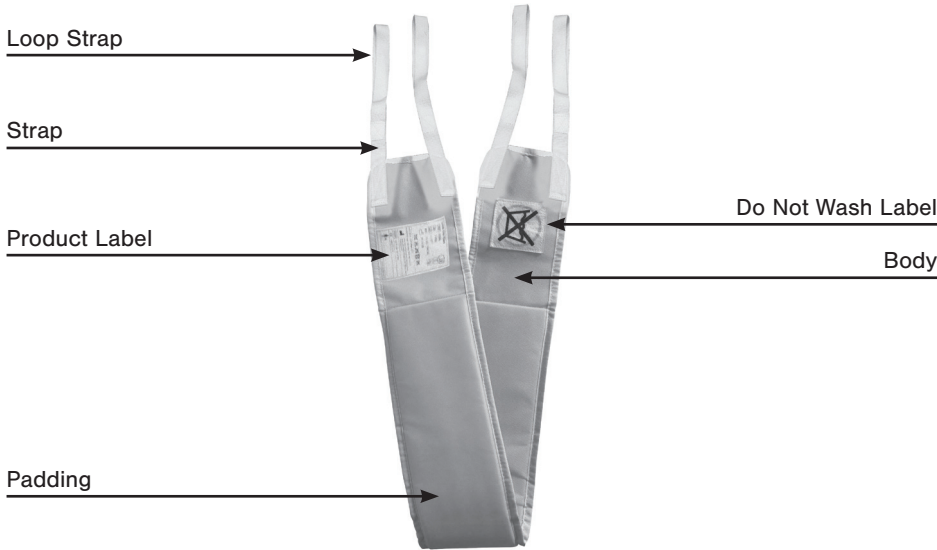

WARNING!

Always read this instructions and the instructions for the lift being used with the sling.

Definition in this IFU:

Warning: Safety warning. Failure to understand or obey this warning may result in injury to you or to others.

Note: Important information for the correct use of this system or equipment.

Safety Instructions:

- Always read the label and check that the sling is in good condition without signs of wear and tear such as loose seams, frays and material inconsistency.
- Always make sure not to exceed the Safe Working Load (SWL) defined for the product.
- Always read the IFU for the lifting equipment.

Warning: Do not use the product if there is any signs of deterioration.

Do Not Wash - this product is designed for single patient use only and should be disposed of if in contact with liquid of any kind.

Safe Working Load(SWL):

70 kg. / 154 lbs.

Intended Use:

The Limb Sling is designed to assist the caregiver for lifting and supporting limbs of patients with reduced mobility, when positioning the limb to perform various tasks, such as: foot care, wound care, venous wrapping etc.

Expected service life:

The service life is expected to be 56 performed lifting sessions, however this will be subject to the actual conditions during use.

The Limb Sling is disposable and designed for single patient use and must be disposed of after patient hospital stay or when soiled or subjected to liquid.

Before and After Every Use:

1. Always perform a risk assessment before starting the specific lifting session in order to make sure the product model is correct and that the lifting procedure is fully understood.
2. Before using the Limb Sling, read this IFU and the IFU for the lifting equipment.
3. When soiled or stained, the sling must be replaced immediately.
4. Check the Limb Sling for any fraying, loose stitching, attachments or any deterioration.
5. **Warning:** Remember to write the patient's name on the care label in order to avoid cross contamination between patients.
6. After each use, check sling according to steps 3 and 4 above
7. Store in a dry area.

Service & Maintenance:

The Limb Sling must be checked by authorized personnel every 6 months according to ISO10535.

Storing:

The sling should be stored in a dry and clean environment. Store at temperatures between (+32°-104°F)

Product care & Maintenance:

- Always follow the instructions on the product label and check the sling for deterioration.
- Mark the sling with the specific patient name and keep in dry and clean environment.
- This is a patient specific disposable sling, that must be disposed after the patient has been discharged.

Attachments:

The Limb Loop Straps will fit most lifts standard Sling bars with loop attachment hooks. Check the Lift manufacturers documentation for details.

Customer Support/Trouble

Shooting:

Please refer to the IFU for the lifting equipment used.

If you have questions please contact Wy' East Medical about the product.

(503) 657-3101



52604-R00

The Limb Sling is designed to lift limbs or a pannus to assist with elevation and treatment of wounds.

Applying the Limb Sling

1. Fold the limb sling and carefully place under the patient's limb. (If on a mattress, push down into the mattress to place sling under patient). Label should be on outside, away from patient.
2. Place sling loops onto sling bar.
3. Carefully raise the sling bar to desired position.
4. Lower until sling straps are loose and remove from under patient.